Clearsight Deluxe Nasal CPAP Mask

Item# **CM1105** Small
CM110M Medium
CM110L Large

Intended Use

The Sunset Clearsight Deluxe Nasal CPAP Mask is intended to be used in home, hospital, or institutional environments for patients who have been prescribed CPAP/VPAP therapy. This device is intended to be used under the specific direction of a physician.

Caution: Federal law restricts this device to sale by or on the order of a physician.

Warnings

- This mask should only be used with CPAP systems recommended by your physician or respiratory therapist.
- Do not use this mask unless the CPAP or VPAP system is turned on and operating properly.
- Never block the mask's vent holes or exhalation port.
- Note: CPAP systems are intended to be used with special masks with ventholes in either the connectors or the mask that allows continuous flow out of the mask. When the CPAP machine is turned on and functions properly, new air from the CPAP machine flushes the exhaled air out through the mask's exhalation port. However, when the CPAP machine is not operating, enough fresh air will not be provided through the mask and exhaled air may be rebreathed. Rebreathing of exhaled air for longer than several minutes can in some circumstances lead to suffocation.
- If oxygen is used with the CPAP system, the oxygen flow must be turned off when the CPAP machine is not in operation.
- Note: When the CPAP device is not in operation and the oxygen flow is left on, oxygen delivered into the ventilator tubing may accumulate within the CPAP machines enclosure and create a risk of fire.
- Note: At a fixed flow rate of supplemental oxygen, the inhaled oxygen concentration will vary depending on the pressure setting, patient pattern and leak rate.
- Do not smoke if oxygen is in use.
- Contact a health care professional if chest discomfort, shortness of breath, stomach distention, belching, or severe headache are experienced.
- Caution: At low CPAP pressures the flow through the exhalation port may be inadequate to clear all exhaled gas from the tubing and some rebreathing may occur.
- Must only be used by persons who have received adequate training and instruction from qualified medical personnel.


Contraindications

This mask may not be suitable for patients with the following conditions: impaired cardiac sphincter function, excessive reflux, impaired cough reflex and hiatal hernia. It should not be used if the patient is unresponsive, obtunded, unable to remove the mask, or uncooperative.

Cleaning instructions

- Detach headgear from mask.
- Handwash gently with mild soap and warm water, rinse the mask and components thoroughly.
- Let mask and components air dry before reusing.
- Replace mask if there are cracks, tears or excessive discoloration.
- Caution: The use of bleach, chlorine, alcohol or aromatic-based solutions may cause hardening of the materials of construction and reduce the life of the product.

Instructions for use

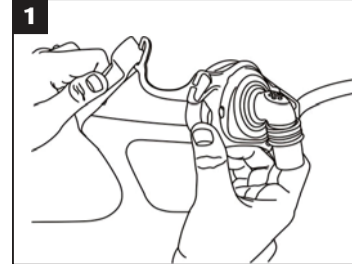Assemble headgear in each curve clip

Place the headgear over your head and adjust both upper headgear straps to a comfortable tension

Attach buckle of bottom headgear straps in mask frame

Fasten bottom headgear straps with velcro side to a comfortable tension

To remove the mask and headgear, simply unbuckle both buckles from the mask frame and pull the upper head strap away from the head.
To reuse the mask and headgear, simply place upper headstrap over head and hook up to both buckles.

Manufactured for
Sunset Healthcare Solutions
180 N Michigan Ave Ste 2000
Chicago, IL 60601

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