



# X-PLO<sub>2</sub>R Maintenance

- Always store the X-PLO<sub>2</sub>R in a cool, clean, and dry area. Exposure to extreme temperatures for an extended period can damage the concentrator.
- Do not store the X-PLO<sub>2</sub>R in an environment where dust can build up and damage the concentrator.
- What should I do to maintain my oxygen equipment?
  - X-PLO<sub>2</sub>R & RECHARGEABLE BATTERY
    - The exterior shell of the X-PLO<sub>2</sub>R and the rechargeable battery should be cleaned as needed.
      - To clean the exterior of the X-PLO<sub>2</sub>R: wipe the outside surface of the case with a cloth dampened with a mild liquid detergent and water solution at room temperature.
      - To clean the exterior of the Battery: wipe the outside surface of the case with a cloth dampened with a mild liquid detergent and water solution at room temperature.
- NASAL CANNULA
  - Replace your supply tubing and cannula on a regular basis as recommended by your equipment provider. Your physician or equipment provider should provide you with cleaning, disinfection and replacement information. Refer to the original manufacturer's instructions for cleaning the nasal cannula.
- CARRYING BAG, HANDLE & STRAP
  - If necessary, you may clean the carrying bag with a damp cloth and a mild liquid detergent and water at room temperature.
    1. Remove the X-PLO<sub>2</sub>R from the carrying bag, if applicable.
    2. Dampen a cloth in a mild liquid detergent and water solution at room temperature and wipe the outside surface of the bag.
    3. Allow time for the bag to dry before placing the X-PLO<sub>2</sub>R back inside.
- AC & DC POWER SUPPLY CORDS
  - The AC and DC power supplies should be cleaned as needed.
    1. Disconnect cords from the POC before cleaning.
    2. Disconnect the power supply from the AC or DC power source.
    3. Clean the cords using a mild liquid detergent and water solution at room temperature and wipe the outside surface.
    4. Allow cords to dry thoroughly before using them again.
- AIR INTAKE FILTER
  - The air intake filter **MUST BE CLEANED WEEKLY** to ensure the ease of air flow.
    1. Remove the filter from the side of the POC.
    2. Clean the filter with a mild liquid detergent and water at room temperature, rinse in water at room temperature and dry before reuse.