




Instruction Manual

Low Concentration Lynx Venturi Oxygen Delivery System

REF RES2400

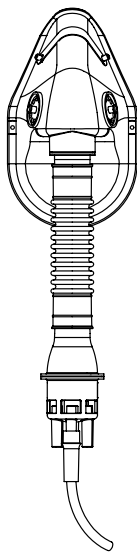

Sunset Healthcare Solutions
141 W Jackson Blvd Ste 1950
Chicago IL 60604
www.sunsethcs.com
MADE IN CHINA

IFU-00085 REV 1.01
2024-08-26

Symbol Glossary



Caution: U.S.A. federal law restricts this device to sale by or on the order of a physician or licensed practitioner.



Not made with natural rubber latex



Do Not Sterilize • Clean, Ready to Use



Single Use



Expiration Date



Consult Instructions for Use



Contains Phthalates



Manufacturer

Cautions and Indications for Use

Caution: Re-processing of products intended for single use only, may result in degradation in performance or loss of functionality. Re-use of single use only products may result in exposure to viral, bacterial, fungal or prionic pathogens. Validated cleaning and sterilization methods and instructions for reprocessing to original specifications are not available for these products. This product is not designed to be cleaned, disinfected or sterilized.


Caution: Be sure all connections are secure.

Indications for Use: The Lynx Venturi System is intended to be used to administer oxygen only. The Lynx Venturi System is for single use only.


Instructions for Use (Reference graphic below)

For Oxygen Delivery Only

1. Attach 22mm corrugated tube to vertical port of All-in-1 Mask.
2. Attach Low Concentration Venturi Adapter to lower end of 22mm corrugated tube.
3. Place All-in-1 Mask on patient's face over the nose and mouth and secure with elastic band provided with the mask.
4. Attach oxygen supply tube to oxygen port for desired oxygen concentration.
5. Adjust oxygen flow in accordance with settings on Venturi Adapter and back cover of this guide for desired oxygen concentration.

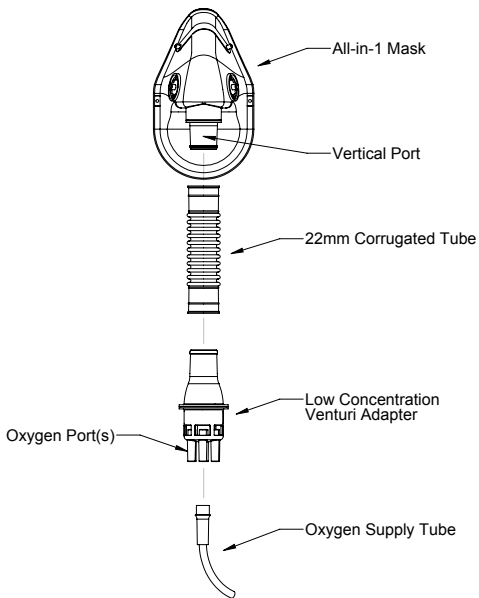

Low Concentration Venturi Adapter

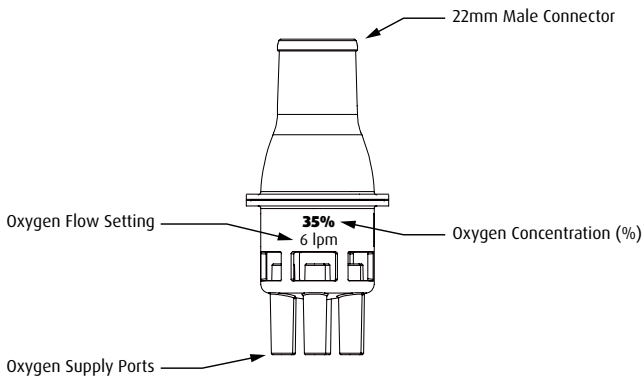

Settings Chart	
FiO_2 (%)	Flow Rate (lpm)
24	3
28	4
35	6
40	8
50	12
60	15