

Final Bacteria Replacement Filter

Product Description and Indications

Sunset Oxygen Concentrator Filters are Replacement Filters intended for use in oxygen concentrator machines to help remove contaminants, including air borne bacteria and other particulate debris from an air stream. When used with oxygen concentrator machines, the replacement filters may be used in the home, nursing home, patient care facility, etc. The Sunset oxygen concentrator filter is a replacement filter for use with Original Equipment Manufacturer's (OEMs) Oxygen Concentrator Machines.

Cautions

1. Federal law (USA) restricts this device to sale by or on the order of a physician.
2. Refer to the OEM's Instructions for Use for instructions on proper installation of filters.
3. Refer to the OEM's Instructions for Use for Recommended Filter Replacement.
4. DO NOT Wash or attempt to clean this filter. Soaking or rinsing of this filter may result in increased airflow resistance.

Warnings

1. Ensure all connections are secure.

REF

Product	Filter Specifications	Machine
BF316	Bacterial Filtration Efficiency (BFE): 99.999+% Connections: 3/16" - 1/4" Stepped Barb	Covidien Nellcor Puritan Bennett 590 and 592A; Respironics Millennium, EverFlo, EverGo; Healthdyne Alliance.
BF516	Bacterial Filtration Efficiency (BFE): 99.999+% Connections: 1/4" Straight Barb	SeQual Integra; Invacare Mobilaire 3 and 5, Platinum 5 and 10, Perfecto2
BF1438	Bacterial Filtration Efficiency (BFE): 99.999+0/0 Connections: 3/16" Hose Barb	AirSep Newlife, QuietLife, Intensity; DeVilbiss/Drive 525 DS Series, 1025 Series, MC Series, SolAiris V, 505, 515 Series, SolAiris III, 303 Series; Nidek Nuvo, Mark IV, V, V+;



Manufactured for
Sunset Healthcare Solutions
141 W Jackson Blvd Ste 1950
Chicago IL 60604