




Manufactured for  
Sunset Healthcare Solutions  
141 W Jackson Blvd Ste 1950  
Chicago IL 60604

IFU-00038 Rev 2.03  
2024-10-22

## Instruction Manual

# Filter and Check Valve Kit for SoClean 2 CPAP Cleaning Device

### **CAP1007S-KIT**

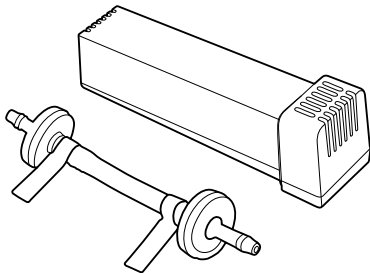



This product is not made, authorized, or endorsed by SoClean, Inc., but is warranted by Sunset Healthcare Solutions, Inc. to fit and work in the SoClean 2 CPAP cleaning device.

## Replace filter



Replace the filter and check valve every six months, when the SoClean prompt shows on the display.

**!** You **MUST** use the filter to prevent harmful unconverted ozone from entering air.

- 1** Remove old filter by pulling up from the filter holster. Dispose of old filter in household trash.
- 2** Completely remove plastic tape from new filter and wipe off any remaining charcoal residue.

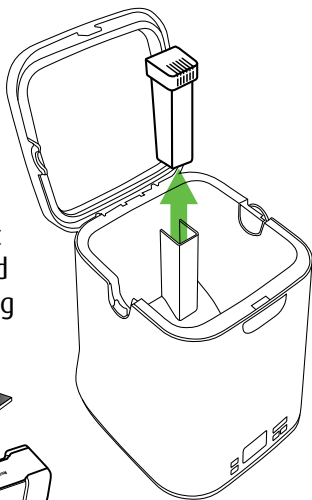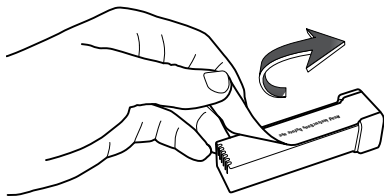


- 3** Place new filter into holster and press down until it is fully seated.

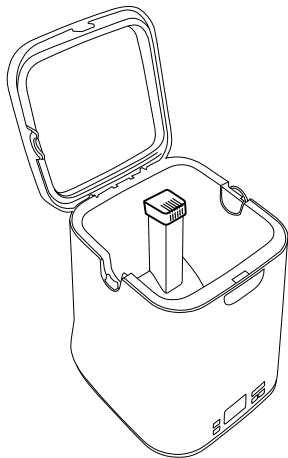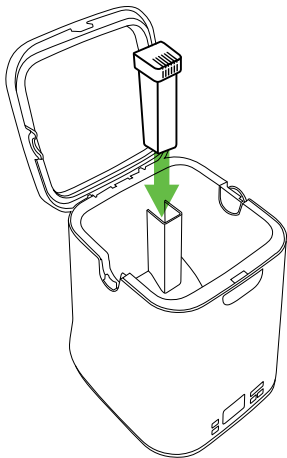


# Replace check valve


- 1** Pull thin black injection tubes off of old check valve.

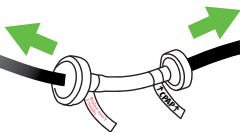

- 2** Put new check valve in place by ensuring firm connections.

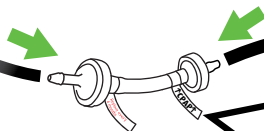

It is important to ensure the **↑CPAP↑** arrows on the check valve are pointed toward the CPAP machine.

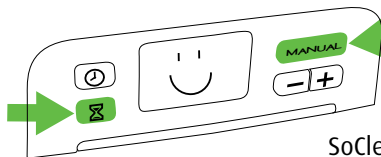

- 3** Reset the replacement timer by holding down the manual and hourglass buttons together until the SoClean shows a smiling face.