- EN** English
- ES** Español
- FR** Français

Instruction Manual

Sol **Full Face CPAP Mask**

CM205

Manufactured for
Sunset Healthcare Solutions
141 W Jackson Blvd Ste 1950
Chicago IL 60604
United States

IFU-00047 REV 1.04
2025-08-20

Intended use

The mask is a noninvasive accessory used for channeling airflow with or without supplemental oxygen to a patient from a positive airway pressure (PAP) device such as a continuous positive airway pressure (CPAP) or bi-level system. The mask is intended for single-patient use. The mask is for users weighing over 66 pounds.

Contraindications

If the user has the following complications, the product may not be suitable for use. Special attention should also be given when it must be used. In either case, it is up to the doctor to decide whether or not to use the mask. These complications include:

- Recent eye surgery or dry eyes
- Glaucoma
- Hiatal hernia
- Impaired cardiac sphincter function, excessive reflux, impaired cough reflex
- Vomiting
- Bullous Lung Disease
- Pathologically Low Blood Pressure
- Bypassed Upper Airway
- Pneumothorax
- Diagnosis of acute sinusitis or otitis media
- Risk of aspiration of gastric contents
- Allergy or hypersensitivity to the mask materials, or inability to remove the mask

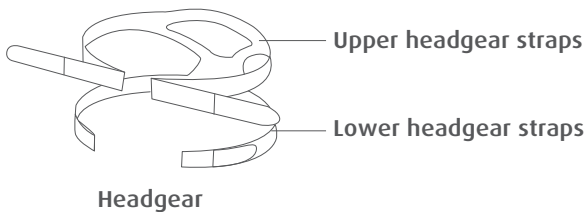
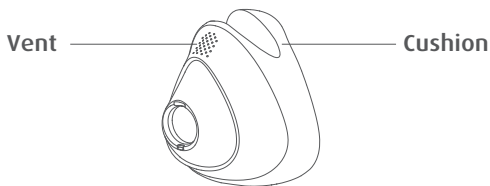
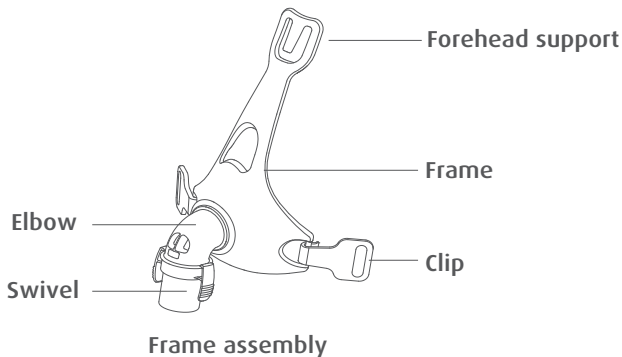
CAUTION:

In the United States, Federal law restricts the sale of this device by or on the order of a physician.

WARNINGS:





- Please read the entire guide before use.
- Remove all packaging before using the mask.
- Ensure mask vent holes are clear from obstruction.
- CPAP systems are intended to be used with special masks with connectors which have vent holes to allow continuous flow of air out of the mask. When the CPAP machine is turned on and functioning properly, new air from the CPAP machine flushes the exhaled air out through the attached mask exhalation port. However, when the CPAP machine is not operating, enough fresh air will not be provided through the mask, and exhaled air will be rebreathed. Rebreathing of exhaled air for longer than several minutes can, in some circumstances, lead to suffocation. This warning applies to most models of CPAP systems.
- At low CPAP pressures the flow through the exhaled port may be inadequate to clear all exhaled air from the tube. Some rebreathing may occur.
- To minimize the risk of vomiting during sleep, the patient should avoid eating or drinking for three hours before using the mask. This mask is not recommended if the patient is taking a prescription drug that may cause vomiting.
- Do not use the mask without the Non-Rebreathing Valve in place.
- This mask should not be used on patients who are uncooperative, obtunded, unresponsive, or unable to remove the mask.
- Follow all precautions when using supplemental oxygen.
- Oxygen flow must be turned off when the flow generator is not operating, so that unused oxygen does not accumulate within the flow generator enclosure and create a risk of fire.
- At a fixed flow rate of supplemental oxygen, the inhaled oxygen concentration varies, depending on the pressure settings, patient breathing pattern, mask, point of application and leak rate.
- The technical specifications of the mask are provided for your clinician to ensure compatibility with the flow generator. If it is used beyond technical specifications or used with incompatible devices, the seal and comfort of the mask may not be effective, optimum therapy may not be achieved, and leak, or variation in the rate of leak, may affect the function of the flow generator.
- Stop using the mask and consult your physician or sleep therapist, if you have ANY adverse reaction to the use of the mask.
- Refer to your flow generator manual for details on settings and operational information.
- As with all masks, some rebreathing may occur at low CPAP pressures.

Parts of the mask



Fitting

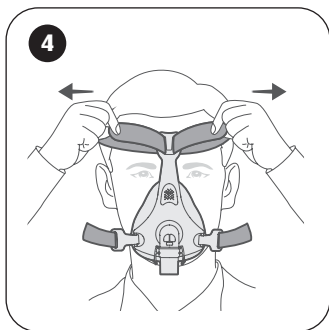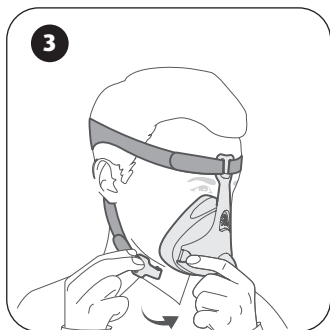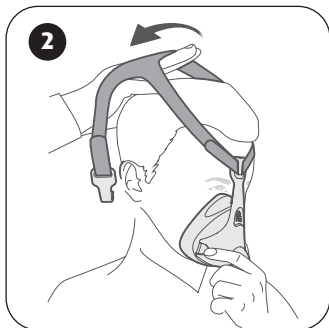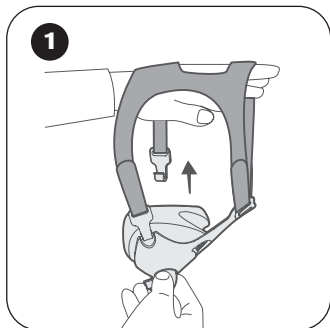Use a standard conical connector if pressure readings and / or supplemental oxygen are required.

Continued on next page

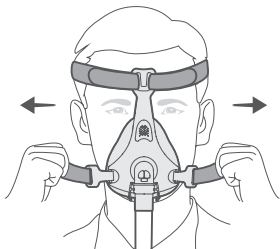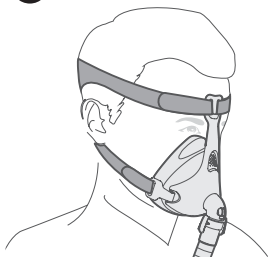5**6**

- 1** Unclip one of the lower headgear clips from the frame.
- 2** Hold the mask steadily on your face and pull the headgear over your head.
- 3** Bringing the lower headgear straps below your ears, clip the headgear to the lower part of the frame.
- 4** Undo the fastening tabs on the upper headgear straps. Pull the straps evenly until they are comfortable and firm, and reattach the Velcro. Ensure that the mask fits well.
- 5** Undo the fastening tabs on the lower headgear straps. Connect the device air tubing to the swivel of mask and ensure it is firmly connected. Pull the straps evenly until the mask is stable and comfortable. Reattach the fastening tabs.
- 6** Your mask and headgear should be positioned as shown in the illustration.

Removal

To remove the mask, disconnect one of the lower headgear clips and pull the mask and headgear up over your head.

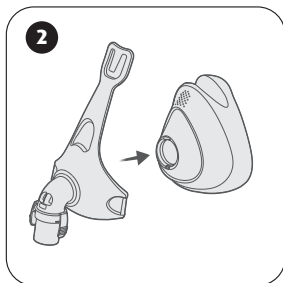
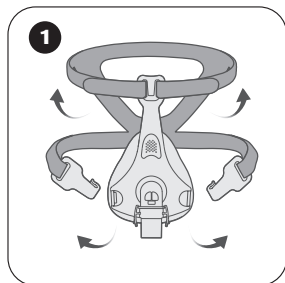
CAUTIONS:

- Please confirm that you can remove the clip and headgear freely
- Do not remove the headgear until the clip is released

Disassembly



⚠ NOTES:

Do not remove elbow assembly unless necessary.

- 1 Detach the air tubing. Unclip the lower headgear clips and unhook the upper headgear straps.
- 2 Gently separate the cushion from the frame.

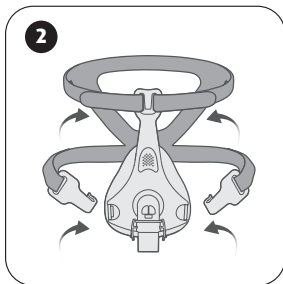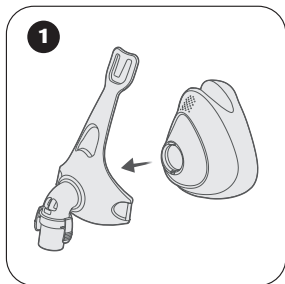Reassembly

- 1 Fit the cushion into the channel of the frame.
- 2 Thread the upper headgear straps through the loop of the frame and secure. Clip the headgear to the lower part of the frame.

Cleaning the Mask at Home

NOTES:

- Only hand wash the cushion, headgear, and mask.
- The elbow and the valve cannot be disassembled for washing.
- If any visible deterioration of a mask component is apparent (cracking, tears, etc), the mask component should be discarded and replaced.
- It is recommended to replace the cushion every month or after 30 cleanings.

CAUTIONS:

- Do not use solutions containing bleach, chlorine, alcohol, aromatics, moisturizers, antibacterial agents, or scented oils to clean any part of the system or air tubing. These solutions may cause damage and reduce the life of the product.
- Avoid exposing any part of the system or tubing to direct sunlight or heat, which may cause deterioration.
- Do not use a dishwasher to wash mask.
- Do not air dry mask components in direct sunlight as it may accelerate product aging.
- Do not iron the headgear. It may damage the headgear.
- If cleaning is not done in accordance with the instructions, or if cleaning frequency is increased, it may result in poor product performance or reduced product life.

Daily / After Each Use

- Disassemble the mask components according to the disassembly instructions.
- Thoroughly clean the separated mask components (excluding headgear), by gently rubbing in warm (approx. 86°F / 30°C) water using mild soap. To optimize the mask seal, facial oils should be cleaned from the cushion after use. Use a soft bristle brush to clean the vent. Soak in water for no more than 10 minutes.
- To optimize the mask seal, facial oils should be cleaned from the cushion after use.
- Rinse all components well with clean drinking water, making sure there is no soap residue. Inspect each component and, if required, repeat washing and rinsing until visually clean.
- Inspect each component and, if required, repeat washing and rinsing until visually clean.
- Allow to air dry out of direct sunlight. When all components are dry, reassemble according to the reassembly instructions.

Weekly


- Hand wash the headgear and all components in warm and soapy water. Use unscented soap. Rinse the components well and allow them to air dry out of direct sunlight, which may age or damage the mask.
- If any visible deterioration of a mask component is apparent (cracking, tears, etc), the mask component should be discarded and replaced.

Troubleshooting

Problem	Possible cause	Solution
Mask is too noisy	Vents are blocked or partially blocked. Mask is assembled incorrectly.	Clean vents according to Instructions in "Cleaning the Mask at Home". Reassemble mask.
Mask leaks around the face	Mask may have been fitted incorrectly.	Carefully follow instructions in "Fitting".
Mask is uncomfortable	Mask may have been fitted incorrectly.	Carefully follow instructions in "Fitting".
Mask is difficult to breathe through	Mask cushion may be dirty.	Clean vents according to Instructions in "Cleaning the Mask at Home".

Technical Specifications

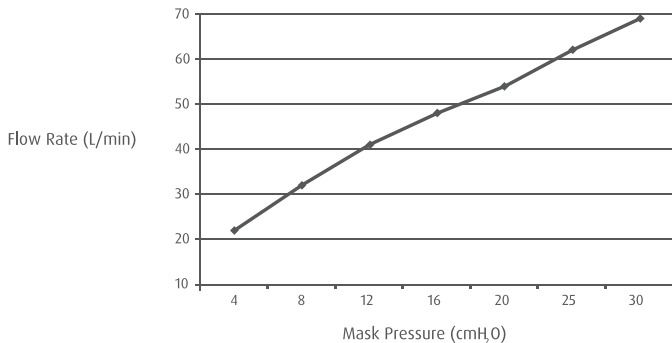Pressure-Flow Curve

Dead Space Information

Dead space is the empty volume of the mask up to the swivel. The dead space of the mask varies according to cushion sizes but is less than 208 mL.

Therapy Pressure

4 to 30 cmH₂O

Resistance

Drop in Pressure measured
at 50 L/min: ≤ 0.5 cmH₂O
at 100 L/min: ≤ 1.5 cmH₂O

Inspiratory and expiratory resistance with the Non-Rebreathing Valve open-to-atmosphere pressure

Inspiratory at 50 L/min: ≤ 2 cmH₂O
Expiratory at 50 L/min: ≤ 2 cmH₂O

The Non-Rebreathing Valve open-to-atmosphere pressure

≤ 2 cmH₂O

The Non-Rebreathing Valve closed-to-atmosphere pressure

≤ 2 cmH₂O

Sound	DECLARED DUAL-NUMBER NOISE EMISSION VALUES in accordance with ISO 4871. The A-weighted sound power level of the mask is less than 38 dBA, with uncertainty 3 dBA. The A-weighted sound pressure level of the mask at a distance of 1 m is less than 30 dBA, with uncertainty 3 dBA.
Environmental Conditions	Operating temperature: 41°F to 104°F (+5°C to + 40°C) Operating humidity: 10%~90% relative humidity non-condensing Storage and transport: -4°F to 140°F (-20°C to +60°C) Storage and transport humidity: 10%~90% relative humidity non-condensing

Note: The mask system does not contain natural rubber latex, PVC or DEHP materials.

Storage

Ensure that the mask is thoroughly clean and dry before storing it for any length of time. Store the mask in a dry place out of direct sunlight.

Disposal

The mask does not contain any hazardous substances and may be disposed of with your normal household refuse.

Symbols

	Batch code
	Date of manufacture
	Catalog number
	Not made with natural rubber latex
	Temperature limitation
	Humidity limitation
	Manufacturer
	Medical Device
	Use-by date
	Consult instructions for use
	Single Patient - multiple use
	Caution, consult accompanying documents. Indicates a Warning or Caution and alerts you to a possible injury or explains special measures for the safe and effective use of the device.
	Unique device identifier

Limited Warranty

It is warranted that the Mask, including Frame assembly, Cushion, Headgear shall be free from defects in material and workmanship for a period of thirty (30) days from the date of purchase by the initial consumer.

This warranty does not cover damage caused by accident, misuse, abuse, negligence, alteration, failure to use or maintain the Product under conditions of normal use and in accordance with the terms of the product literature, and other defects not related to materials or workmanship.

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