Classic Full Face CPAP Mask

Item# **CM007S** Small **CM007L** Large
CM007M Medium **CM007XL** Extra Large





Intended Use

The Sunset CPAP Mask channels airflow noninvasively to the patient from a positive airway pressure device such as a continuous positive airway pressure (CPAP) or bilevel system. The Sunset CPAP Mask is to be used by adult patients for whom positive airway pressure has been prescribed. It is intended for use in the home environment or the hospital/institutional environment.

Caution: Federal law restricts this device to sale by or on the order of a physician.

⚠ Warnings

- This mask should only be used with CPAP systems recommended by your physician or respiratory therapist.
- The Sunset Full Face mask must be used under qualified supervision with patients who are unable to remove the mask by themselves. The mask may not be suitable for those with the following conditions: impaired cardiac sphincter function, excessive reflux, impaired cough reflex and hiatal hernia.
- All vent holes and the exhalation port must be kept clear.
- Do not use this mask unless the CPAP or bilevel system is turned on and operating properly. CPAP systems are intended to be used with masks with vent holes that allow continuous flow out of the mask. When the CPAP device is turned on and functions properly, new air from the CPAP machine flushes the exhaled air out through the mask's exhalation port. However when the CPAP machine is not operating, there may not be sufficient fresh air provided through the mask and exhaled air may be rebreathed. Rebreathing of exhaled air for longer than several minutes can cause suffocation.
- Follow all precautions when using supplemental oxygen.
- Any oxygen flow must be turned off when the flow generator is not operating so that unused oxygen does not accumulate within the flow generator enclosure and create a risk of fire.
- At a fixed flow rate of supplemental oxygen flow, the inhaled oxygen concentration will vary, depending on the pressure settings, patient breathing pattern, mask, point of application and leak rate.
- Discontinue using the Sunset CPAP mask if you have ANY adverse reaction to any part of the mask system and consult your physician or sleep therapist.

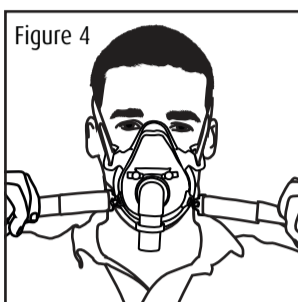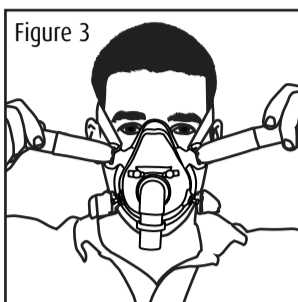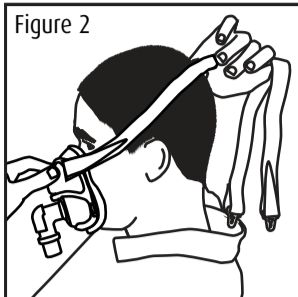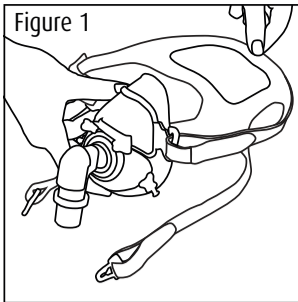We recommend that an assistant help you with the initial fitting of your mask and headgear to ensure proper fit.

Attachment Instructions

1. Attach the top two straps of the headgear to the mask with Velcro. The straps should be as loose as possible to start. Attach the headgear to the clips but detach the clips from the mask so the bottom portion of your headgear remains free (see figure 1).
2. Hold the mask steadily on your face and pull the headgear over your head (see figure 2).
3. Bringing the lower straps below your ears, clip them to the lower part of the mask frame.
4. Unfasten the Velcro on the two top straps and pull the straps evenly until they are comfortably snug (see figure 3), reattach the Velcro.
5. Repeat step 4 with the lower headgear straps (see figure 4).
6. Connect the CPAP tube to the mask and machine, and turn on the device.
7. Lay in your sleeping position and check to make sure the mask feels secure and does not leak. Adjust straps as necessary for a comfortable fit.

Removal Instructions

To remove the mask, disconnect one of the lower straps using a clip and pull the mask and headgear up over your head.

Cleaning Instructions

Daily: Wipe the mask and cushion using a CPAP Cleaning Wipe or hand wash with a mild soap and warm water. Rinse and lay flat to dry.
 Weekly: Hand wash the mask and headgear with a mild detergent in warm water. Rinse thoroughly and lay flat to dry out of direct sunlight.
 DO NOT machine wash, dry or iron the headgear.
 DO NOT use scented soaps or oils, bleach or alcohol to clean any of the mask components as they may leave a residual smell or damage the mask.

Manufactured for
 Sunset Healthcare Solutions
 141 W Jackson Blvd Ste 1950
 Chicago IL 60604

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