

Inline Heat and Moisture Exchanger (HME)

With Gas Sampling Port

Small Volume (70-600mL)



REF RES027IL

Intended Use

The HME is disposable single-use device indicated for patient requiring humidification during the delivery of ventilator gases. The HME is for use in hospital, ICU, anesthesia, respiratory therapy, during transport and with resuscitators. The HME can be used on pediatric patients. The HME is indicated for use by qualified medical personnel only. The HME is designed for disposable use and it is recommended to be changed at least every 24 hours, or as recommended by a physician.

Technical data

Moisture Output	34.5 mgH ₂ O/L at VT = 250 ml
	32.5 mgH ₂ O/L at VT = 500 ml
Recommended tidal volume	70-600 ml
Resistance	30L/min: 0.8 cmH ₂ O
	60L/min: 2.1 cmH ₂ O
Dead Space	21 ml
Moisture Loss	(per ISO 9360 test procedure)
	Tidal Volume = 250 ml 9.3 mg H ₂ O/L
	Tidal Volume = 500 ml 11.3 mg H ₂ O/L
Weight	9g
Connections	22M/15F-15M



Instructions for use

Read these instructions carefully before using the product.

Place the HME between the proximal end of the artificial airway and the Y-piece of the breathing circuit.

Always replace the HME after each patient.

When used continuously on a single patient, it is recommended to change the HME every 24 hours or more frequently as required, or as recommended by a clinician.

Contraindications

The HME is contraindicated in patients producing fulmination, frothy secretions within their airway and lungs.

The HME shall not be used on patient with very small tidal volumes, for example neonates.

The HME shall not be used together with active humidifier or nebulizers.

Precautions

All tubing and connections to the HME shall be properly attached and checked for leakage prior to use.

Before use, verify that the HME has no occlusions and that air will flow through it.

Compensation of ventilation may be necessary when using the HME since dead space will be added to the system.

During the use of the HME, the patient shall be closely monitored and proper airway care administered if complications arise.

If a sampling tube is not connected, make sure that the sampling port cap on the HME is properly secured.

The HME must be changed between patients.

Warning

The HME is designed for single patient use only and must not be cleaned and reused.

Storage

Store in a clean, cool, dry place in original wrapper.

Shelf life: Max. 3 years. Expiration date printed on the package.

Manufactured for
Sunset Healthcare Solutions
141 W Jackson Blvd Ste 1950
Chicago IL 60604
www.SunsetHCS.com

MADE IN CHINA

IFU-00118 REV 1.03
2026-06-22